## Safety Communication

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#### Concerns/Potential risk:

Patients might be infected with Covid-19 and the tests shows negative results.

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#### Recommendations for healthcare providers:

1. Antibody rapid diagnosis tests shall **NEVER** be considered as a replacement for PCR, as these tests show low sensitivity in the early detection of COVID-19. However, these tests are complementary and serve different purposes.
2. Antibodies rapid test for COVID-19 can be used for disease surveillance and epidemiological studies, to identify cases which were not known to be infected either because they never developed symptoms or had the symptoms but were never diagnosed”.
3. Most studies showed that COVID-19 antibodies can be detected in the blood 10-14 days after the infection or symptoms. Please follow the manufacturer recommendation in this regard.
4. This test can help healthcare professionals to identify individuals who have overcome an infection in the past and have developed an immune response.

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If there is any complain or need further information, please report the problem to SFDA through:

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