

Safety Communication

رسالة سلامة

Potential Cybersecurity Vulnerabilities to SpaceCom, Battery Pack SP with WiFi, Data Module Compactplus

Device/ Product Description:	Network communication devices
Affected product:	SpaceCom, Battery Pack SP with WiFi, Data module compactplus
Manufacturer:	B. Braun Melsungen AG
Problem:	<p>B. Braun was made aware of potential cybersecurity vulnerabilities in the following products:</p> <ul style="list-style-type: none"> • SpaceCom, software versions L81 and earlier • Battery pack with WiFi, software versions L81 and earlier • Data module compactplus, software versions A10 and A11 <p>Under certain conditions, successful exploitation of these vulnerabilities could allow a sophisticated attacker to:</p> <ul style="list-style-type: none"> • Compromise the security of the Space or compactplus communication devices, • Escalate privileges, • View sensitive information, • Upload arbitrary files and perform remote code execution or change the configuration of a connected infusion pump Perfusor, Infusomat P from both Space and compactplus family which may alter infusions after a successful attack.

<p>Recommendation /Actions:</p>	<ul style="list-style-type: none"> • Update SpaceCom software version to L82 or later versions. • Update Battery Pack SP with WiFi to L82 or later version. • Update Data module to version A11 (I0050A0011) or later. <p>For more information, please click here.</p> <p>If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through: NCMDR Vigilance system (19999)unified call center</p>	
<p>Authorized Representative Details</p>	<p>AR name:</p>	<p>Medical supplies and services Ltd.Co</p>
	<p>Assigned Contact Person:</p>	<p>Bader Al Shabrain</p>
	<p>Mobile/Phone:</p>	<p>0535000220</p>
	<p>Email:</p>	<p>b.alshabrain@mediserv.com.sa</p>

