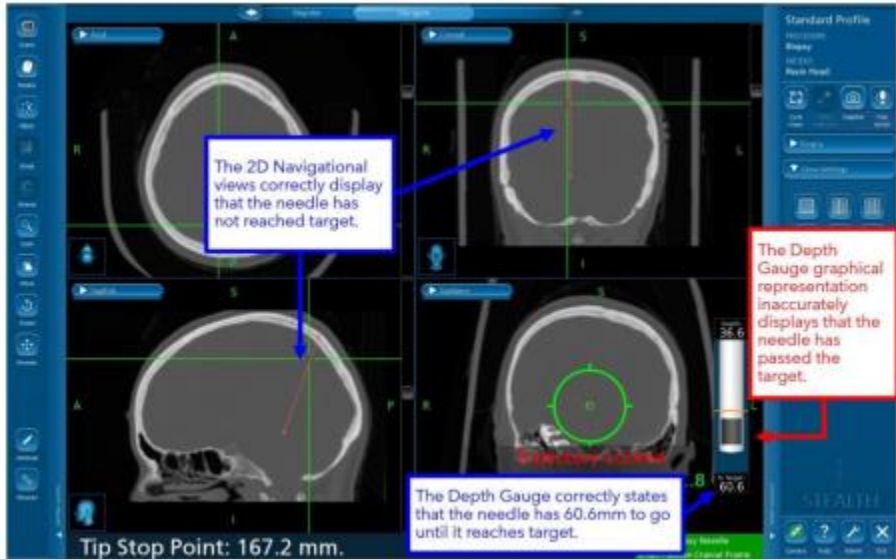


Safety Communication

رسالة سلامة

Potential Inaccuracy During Biopsy Procedures

Device/ Product Description:	Synergy Cranial S7 and StealthStation Cranial			
Affected product:	Navigation System	Software Name	Model#/CFN	Version
	StealthStation S7/i7	Synergy Cranial S7	9733763	2.2.8
	StealthStation S7/i7	StealthStation Cranial	9735585	3.1.1
	StealthStation S7/i7	StealthStation Cranial	9735585	3.1.2
	StealthStation S7/i7	StealthStation Cranial	9735585	3.1.3
Manufacturer:	Medtronic Navigation, Inc.			
Problem:	<p>The software can enter a state where the Biopsy Depth Gauge is no longer synchronized with the rest of the navigational information on the screen and displays an inaccurate position of the biopsy needle.</p> <p>This issue can potentially lead to resection of normal brain tissue or eloquent anatomical regions of the brain. When this software anomaly is encountered, it may result in a prolonged procedure, the need for an additional surgical procedure, tissue injury, including potential for life-threatening injury (hemorrhage, unintended tissue damage, permanent neurological injury).</p>			

<p>Recommendation /Actions:</p>	<p>Medtronic will provide a warning and instructional placard to be applied to impacted systems to maintain visibility to the mitigations until a software fix is available. Please attach the warning and instructional placard to your impacted StealthStation systems. A Medtronic representative can assist you with placing the placards.</p> <p>The Mitigation Steps should be utilized to prevent this software anomaly from occurring and to restore normal function if this anomaly is observed.</p> <p>For more information, please click here.</p> <p>If you think you had a problem with your device or a device your patient uses, please report the problem to SFDA through: NCMDR Vigilance system (19999)unified call center</p>	
<p>Devices/Products photo:</p>		
<p>Authorized Representative Details</p>	<p>AR name:</p>	<p>Medtronic Saudi Arabia LLC</p>
<p>Assigned Contact Person:</p>	<p>Faisal Matbuli</p>	
<p>Mobile/Phone:</p>	<p>+ (966) 535355500</p>	
<p>Email:</p>	<p>ksa.ra@medtronic.com</p>	