

# Differences between FlexPen® and FlexTouch®

	FlexPen®	FlexTouch®
Usage	Pre-filled	Pre-filled
Single vs multi-use	Multi-use	Multi-use
Dosing	Variable	Variable
Min/max dose (insulin)	1 -60	1 –80
Activation mode	Push	Push
Extension button	✓	✗
Click at end of dose	✗	✓
Dose increment	1 unit per click	1 unit per click
Needle gauge	Up to 32G	Up to 32G
Cartridge size	3 mL	3 mL
Weight with cap (approximately)	24 g	33 g
Click during dose setting	✓	✓
Dose dial clicks forward and backward	✓	✓
Dials back to zero during injection	✓	✓
Dose Accuracy	Similar <sup>1</sup>	Similar <sup>1</sup>
Mean Injection force	13.0±0.61 N <sup>1</sup>	5.1±0.48 N <sup>1</sup>
Easy to read dose display	✓	✓
Color differentiation	✓	✓
Compatible with most needles	✓	✓



Pen includes feature



Pen does not include feature



FlexPen®, FlexTouch® are registered trademarks of Novo Nordisk A/S.  
<sup>1</sup>. Wielandt JO, Niemeyer M, Hansen MR, et al. An assessment of dose accuracy and injection force of a novel prefilled insulin pen: comparison with a widely used prefilled insulin pen. Expert Opinion on Drug Delivery. 2011;8(10):1271-1276