**Introduction:**

In accordance with the National Center for medical devices reporting efforts in investigating medical devices adverse events and recalls, including reports and alerts received from other international regulatory authorities, global reporting centers and medical devices manufacturers and importers. The national center for medical devices reporting received a recall from **LifeScan** establishment concerning **One Touch Sure Step test strip** (see appendix.1).

**Nature of the problem:**

The OneTouch SureStep devices are designed to provide numeric glucose results between 0 and 500 mg/dL (27.8 mmol/L).

Some lots of OneTouch Test Strips may provide lower test results when a patient's blood glucose value is greater than 500 mg/dL (27.8 mmol/L).

The meters are designed to display 'High' or "HI" when the meter glucose result is above 500 mg/dL (outside of the labeled dynamic range), it has been observed that after testing with reference glucose of 550 mg/dL (30.6 mmol/L), its meter did not display High and instead provided a faulty numerical result.

This issue may contribute to lower blood glucose results at high blood glucose values, which can lead to a patient being undertreated for hyperglycemia.

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**Recommendations.**

- **SFDA recommend Hospitals and Healthcare Professional Customers with the following :**
- Assure getting **OneTouch SureStep strips** related information details from **LifeScan** representative in the kingdom Banaga medical equipmet company , in addition getting instructions on how to continue to use the product safely and effectively.
- Instruct all medical staff to confirm via an alternate means or a lab test any meter result in excess of 400 mg/dL. If the ability to confirm results is not readily available, the recommendation will be to guide preliminary treatment by conducting a clinical assessment of the patient.
- Adjust their “Critical High” protocol threshold to no more than 400 mg/dL.
- Do not dispense any SureStep test strips to the patients to be use at home.

**This Safeguard notification has been circulated to all hospitals and healthcare providers in Saudi Arabia.**

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